



Quick Start:

Installation & Setup for "Single-User" (stand-alone PC) configuration

Step 1: Install DiaTrends Version 4.1.36

- Close any programs that are running (including antivirus and utility programs, Internet connections, etc.).
- Insert the DiaTrends® version 4.1.36 Installation CD into your CD or DVD drive.
- InstallShield® should automatically start. (If not, open Windows Explorer, browse to the CD or DVD drive and double-click the file "Setup.exe"). Please be patient through any apparent pauses.
- Answer the InstallShield wizard questions and follow the instructions on your screen.
- In the "Setup Type" box, be sure to select "Single-User."
- Continue with the on-screen instructions, simply accepting the defaults.
- Click "Install." Please be patient while files are being installed.
- Click "Finish." Restart your computer if prompted to do so.
- Remove the CD when entire install process (including restart, if prompted) is done.

Step 2: Install the License

- Click the "DiaTrends 4.1" desktop icon. Log in with:
 - User Name: admin
 - Password: diatrends
- The first time you start DiaTrends, you will get "this product is not licensed..." and you will be asked if you have a License Code to enter. Click "Yes" to open the "License Manager."
- Carefully enter your "Serial Number" (4 digits), your alphanumeric case-sensitive "License Code" (26 character) and "Validation Key" (8 character) exactly as shown on your *Installing Your DiaTrends License* document. Then click "OK."
- You should see a message confirming that your license was successfully deployed.

Step 3: Log In and Complete Setup Tasks

- When asked "would you like to continue loading DiaTrends?" click "Yes."
- Follow on-screen instructions for setup tasks.
- Each user should enter a unique user name, and each user (including "admin" which is a permanent user name) should change his or her password (see instructions in the user guides and embedded "Help" files).
- Please remember you must enter a physician name (and specify "Physician" in the "Provider Type" drop-down list) before you can begin entering patient data. DiaTrends will automatically open to the "Providers" screen until a physician name has been entered. Or, go to "View → Options → Providers" to set this up.